

LCD skin cleaner operation instruction



Shenzhen Bowei tech Co.ltd.

ADD:RM902;ShangMei Times;NO.57;LongGuan RD;Long Hua Dist Shenzhen, China;518000
Tel:86-755-29994192 13148703639
Email:sales@bowei-ch.com <http://www.bowei-ch.com/>




LCD Skin Cleaner



LCD Skin Cleaner

Part Name:

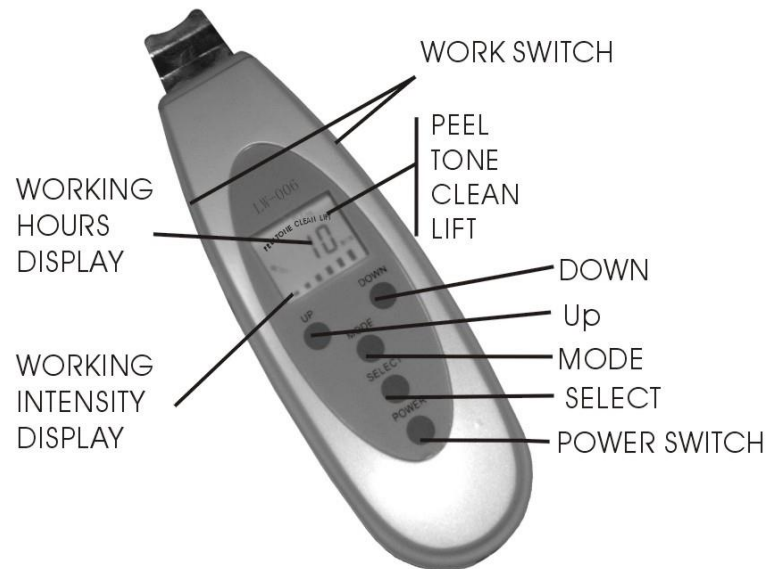

Working principal:

Supersonic skin scrubber is the most popular with beauticians among multiple instruments. After long time's trialing, ensure its outstanding effect on facial care. It shifts electric vibration with 28000 times per second to mechanical vibration with thousands upon thousands of times per second. Penetration effect of ultrasonic gives skin massaging as well as cleansing. Meanwhile, deep cells get excited. It could effectively remove spots, wrinkles, pimples, horniness layer and residues in pores so that your skin looks white, purified and soften. It can be used at any age, for women and men. It is effective for all stages of life and all types of skin and provides an effective solution with instantly visible results. Matured and dull skin benefits from the antiaging effect due to the exfoliation, which reduces wrinkles and fine lines and stimulates cell regeneration. Dry and sensitive skin is revitalized as aged skin cells are removed with the result that products used after treatment (face masks, creams, etc.) work better. The skin is unblocked and elasticity is restored. Greasy and problem skin

Shenzhen Bowei tech Co.ltd.

ADD:RM902;ShangMei Times;NO.57;LongGuan RD;Long Hua Dist Shenzhen, China;518000
 Tel:86-755-29994192 13148703639
 Email:sales@bowei-ch.com <http://www.bowei-ch.com/>

2

LCD Skin Cleaner

Becomes clean and clear. Pores are closed, the skin texture is refined and imperfections are reduced. It has revolutionized beauty at home with its natural, pure and simple way to contribute to the beauty of your skin and the radiance of your face. By using it, your face has an immediate glow, akin to that of having new skin.

3


LCD Skin Cleaner

Functions and Characters:

1. Deep clear and clean up aged horniness.
2. Reduce melanin to get a whiten skin.
3. In-depth eliminates the wrinkle, strengthens the skin elasticity.
4. Improve blood circulation and lymph metabolism
5. Strengthen the skin cell reproductive property and the permeability.
6. Accelerate the absorption of nutrition.
7. Digital control, with double function integration
8. No effect and negative for skin.
9. Slick design, convenient and easy operation, small volume to keep.


4

LCD Skin Cleaner

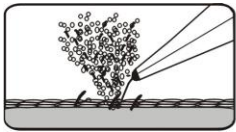


(1)

1, Clean acne and grease on the surface: To clean the acne and grease in pore by the effect of ultrasonic vibration and get the result of radiance of the skin. Take 45 degrees angles as good when operation.

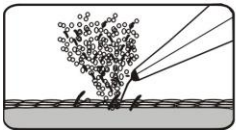


(2)

2, Treat acneic skin: While the pore is blocked, the skin is likely to be acneic. Ultrasonic vibration makes skin surface soften and smooth. Take 45 degrees angles as good when operation.

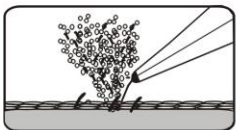


(3)

3, Easy to remove aged horniness: Using 45 degree angle operating procedures, on the skin surface. Excellent effects will be seen immediately.

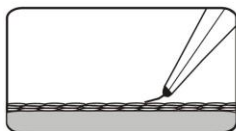

(4)

4, Wrinkles with the degreasing recuperation: Take the advantage of vibration and heat energy of ultrasonic to reduce wrinkles, and stimulates cell regeneration and prevent skin from being aged.

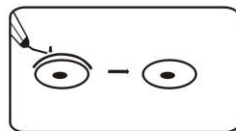

(5)

5, Light and clean out melanin: As vibration of the ultrasonic makes TYROSINASE come out and also give you're a new and whiten skin.

5

LCD Skin Cleaner

Cautions:

- 1, Please use this machine together with clear water, toilet gel or astringent. This may get better effect.
- 2, Massage, wrinkles with the transformed waves or the continuous wav.
- 3, Please keep the probe and mainframe clean and sterilize before and after using.
- 4, It is not allowed to use this machine on area around eyes and wound place.
- 5, Children, pregnant women and people with mental diseases are not allowed to use it.
- 6, The working time can be adjusted according to the request.
- 7, It's normal if there is a sound of "zhizhizhi" while the machine works, as its working frequency is something like sound frequency.
- 8, If the machine is broken and doesn't work, please contact the manufacturer and agent.

6

LCD Skin Cleaner

Item No.:LW--006

Specifications:

- 1,Output frequency:7.0W
- 2,Output voltage: DV--15V
- 3,Dimension:25cmX20cmX7cm
- 4,N.W.:0.610KGS
- 5,G.W.:0.610KGS

7

LCD Skin Cleaner

Product written guarantee

We guarantee the product of our company in guarantee period ,plesse contact with the store when the product have malfunction problem.This written guarantee is guarantees in the free service period.Not included the following situations:

- 1,Has not recorded the shop name and the purchase date.
- 2,Written guarantee losing or does not identify.
- 3,Has not obeyed the operation instruction to use this product or to take apart for overhaul voluntarily.
- 4,Because the factor of falling causes the damage.
- 5,Because the voltage unstable or does not use the factory standard equipment creates the damage.

Free service guarantee time:1 year

Purchased date:

Shenzhen Bowei tech Co.ltd.

ADD:RM902;ShangMei Times;NO.57;LongGuan RD;Long Hua Dist Shenzhen, China;518000
Tel:86-755-29994192 13148703639
Email:sales@bowei-ch.com <http://www.bowei-ch.com/>

bowket

Purchasedshop:

8

Shenzhen Bowei tech Co.ltd.

ADD:RM902;ShangMei Times;NO.57;LongGuan RD;Long Hua Dist Shenzhen, China;518000
Tel:86-755-29994192 13148703639
Email:sales@bowei-ch.com <http://www.bowei-ch.com/>